


## ACS REVIVO SPECIFICATION SHEET

## Revivo Portable Electronics Dryer & Sanitizer

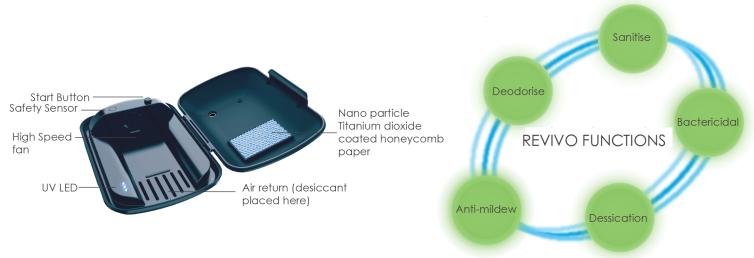

ACS Revivo is a portable electronic device designed to remove the harmful moisture and bacteria that can penetrate into any electronic device worn in the ears. The special high-speed fan circulates air inside the Revivo and capture moisture via the replaceable silica gel desiccant pack. Nano particle Titanium dioxide and UV photocatalytic reactions generate hydroxyl radicals and negative oxygen ions to both sterilize and sanitize the devices.

Program cycles can be set anywhere from 1 to 30 hours to ensure moisture and hygiene issues of the ear worn electronics devices become a thing of the past.

- Removes harmful moisture from earphones and hearing instruments to preserve optimum performance.
- Advanced TiO2 UV photocatalytic reactions sterilize & sanitize your devices for better hygiene
- Simple one-touch operation and setting
- Lithium-ion battery provides up to 30 hours of standard portable operation
- Convenient micro USB charging

## Dry Capsule Desiccant

10 PCS Pack

ACS Dry Capsule is specially selected toprovide Revivo a compact & efficient moisture capture / removal medium to deliver optimum desiccation performance. Each Dry Capsule is factory sealed with atear-off foil strip to ensure the desiccant inside stays fresh until needed. The desiccant in each fresh capsule is colored orange, When it becomes saturated and needs replacement the color will change to dark green. For maximum usage, ziplock bags are provided to keep the opened capsule sealed until next usage. The desiccant inside the Dry Capsule is silica gel and is not edible. Care must be taken to keep the capsules away from children to avoid any choking hazard.







Specifications

Size: 143 x 104 x 48mm Weight: 225g Power: 4400mAH Li-ion Batteries Battery Operation Time: Up to 30 hours Charging Connector Socket: Micro USB Charging Time: 8 hours @ 500mA input

